

Intruccion manual

Gel 008 : transparent, 260ml, bottle

The products must be used by qualified healthcare personnel

Gel ECO is top quality product with excellent ultrasound transmission and probe sliding properties that will remain stable over time. The colorless get it also suitable for IPL applications.

Gel ECO does not oxidize, it does not contain fatty substances, it does not leave permanent residues on probes, and is easy to remove. It contains bactericidal Euxyl® K100 whose active components are Benzyl Alcohol, Methylchloroisothiazolinone, Methylisothiazolinone.

Conforming to ISO 10993-1 standard on biocompatibility.

It does not contain formaldehyde. Will not damage the probes of the echography equipment.

Warnings

For external use only

The product is supplied un-sterilized

Do not use for ECG recording

This is a single-use product, do not –re-use it. Reusing the product may lead to : alteration of materials: loss of initial functional features of product.

Do not apply gel over irritated eyes or skin or with abrasions of wounds

The product may cause imitation of reddening of skin, especially on allergic people.

Do not swallow. By using the gel for IPL-applications, its characteristics can be damaged: therefore once utilized the gel, do not re-use it on other parts of the body.

Close tightly after use. Do not use the gel after the use by date on the bottle

The bags are equipped with a spout for easy pouring of the gel. After pouring out, replace the spoul with the screw cap, for a proper preservation of the gel.

First and measures

Eyes: in case of accidental contact with eyes, rinse profusely with wait for 15 minutes. If irritation persists, consult a doctor.

Skin. If skin becomes irritated, wash thoroughly with water. If irritation persists seek medical attention

Ingestion: Accidental ingestion usually is safe, but consult a doctor, if complications arise.

Instructions for use

People sensitive to MCI/MI should wear protective gloves during the use

Apply the desired amount of GEL ECO

Connect and use the equipment and the probes by following the instructions supplied by the manufacturers

After the procedure remove oall residues of the gel with paper, TNT (Non –woven) or gauze moistened with water

For IPL applications please spread an uniform coat of gel according to the manual furnished together with the IPL-device

Storage

The product must be stored in the original package at the environmental conditions specified on the pouch's label.

Warranty

Compliant with current European legislation on Medical Devices Manufactured according to the procedures of the ISO 13485 certified quality system. No responsibility can be attributed to compensate of medical expenses or direct or indirect damages, deriving from the non-functioning or anomalies of the above devices if they are used differently than a foreseen that in these instructions for use. It is recommended to promptly inform the Manufacturer's and to the competent authority of the Member Stage where the user and / or patient is located.

Waste Disposal

The waste coming from the sanitarian structures must e discharged according to the regulation in force. This gel is water-soluble and is no cause of environment pollution. The container can be given to the diversified disposal

The text is coming from the Manual of FIAB for east reading .